ultraMEDIC[®]

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 Page 5

X-BOON TWO

Vakuummatratze

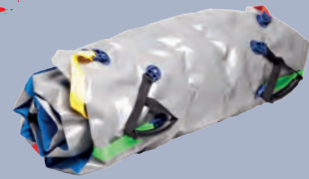
www.ultramedic.de

X-BOON TWO

Vakuummatratze

READY TO USE

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READY TO USE

✓ **READY FOR IMMEDIATE USE**

The point chamber system ensures optimal distribution of the granules. A pre-modeling is therefore no longer necessary. The X-BOON TWO is immediately ready for use.

- space-saving storage
- easy transport in the backpack carrying system
- specific distribution of the chambers (only possible in a point chamber system)
- asymmetrical chamber pattern
- granules in sufficient quantity at only 6.3 kg weight

✓ **CARRY & SECURE: FLEXIBLY ADJUSTABLE**

Familiar color coding of the patient straps, which can be easily moved and removed, as well as the carrying handles, e.g. to

- shift the center of gravity when carrying or securing
- attach additional belt or handle
- clean
- replace a defective belt

... and that with one hand in a few seconds.

✓ **SURFACES**

Lying surface made of skin-friendly TPU material without plasticizers, nevertheless

- insensitive to UV influence
- more abrasion resistant than PVC
- resistant to a large number of solvents → beneficial for disinfection

ROLL & RESCUE

Rollable - thanks to point chamber system

The *point chamber system* is a further development of the multi-chamber system, in which a large number of vacuum chambers ensure, that the granules are distributed more evenly.

The optimum distribution of the granules ensures stability. This means that no additional stabilizing elements are required, which prevent the mattress from rolling up.

The special feature of the point chamber system is a slightly offset, asymmetrical arrangement of the chambers and the targeted placement of more or less granules at different regions of the body, e.g. more granules at the pelvis and spine and less on the legs. Nevertheless, there is no displacement of the granules at any granules at any point, so support is always guaranteed.

Due to more than 140 individual vacuum chambers the granules cannot shift or collect in one place, as it is the case, for example, with a beanbag. The mattress is immediately ready for use after or rolling out without shaking or modeling.

Fastening of the handles and straps

Tool-free and in one movement both can be attached to the permanently mounted plastic mushroom heads and released again, depending on the application required.

The handles hold a weight that is significantly above the required weight from the standard. At significantly higher load, the handles may slip out of the anchorage (approx. from 100 - 120 kg per handle). However, they are not damaged, but can be immediately be hooked back into the mushroom head. This procedure was specially developed to prevent the mattress from excessive tensile load.

Roll instead of fold

Of course, the X-BOON TWO can be folded as usual and stored in the storage compartments or the matching bag. As an interesting variant, however, it offers the possibility to roll it up and store it in a in a backpack carrying system that can be easily transported to more distant locations.

ultraROLL BAG

X-BOON
Carrying bag

X-BOON TWO Vacuum Mattress

Art. No. SAN-0260-2

kg 6.3 kg

Dimensions 200 x 92 cm

kg max 150 kg

§ DIN 1865

MRI and X-ray compatible

Accessories

Vacuum pump: SAN-0278

Storage: ultraROLL BAG
SAN-9000-T
Carrying bag
SAN-0260-T

Spare parts: **BELTS**
SAN-0260-2-G1 *blue*
SAN-0260-2-G2 *yellow*
SAN-0260-2-G3 *green*
SAN-0260-2-G4 *red*
SAN-0260-2-G5 *black*
SAN-0260-2-G6 *Head strap*

HANDLE
SAN-0260-2-GR

REPAIR SET
SAN-0260-2-REP

ultraMEDIC
Im Bruch 11-15
56567 Neuwied